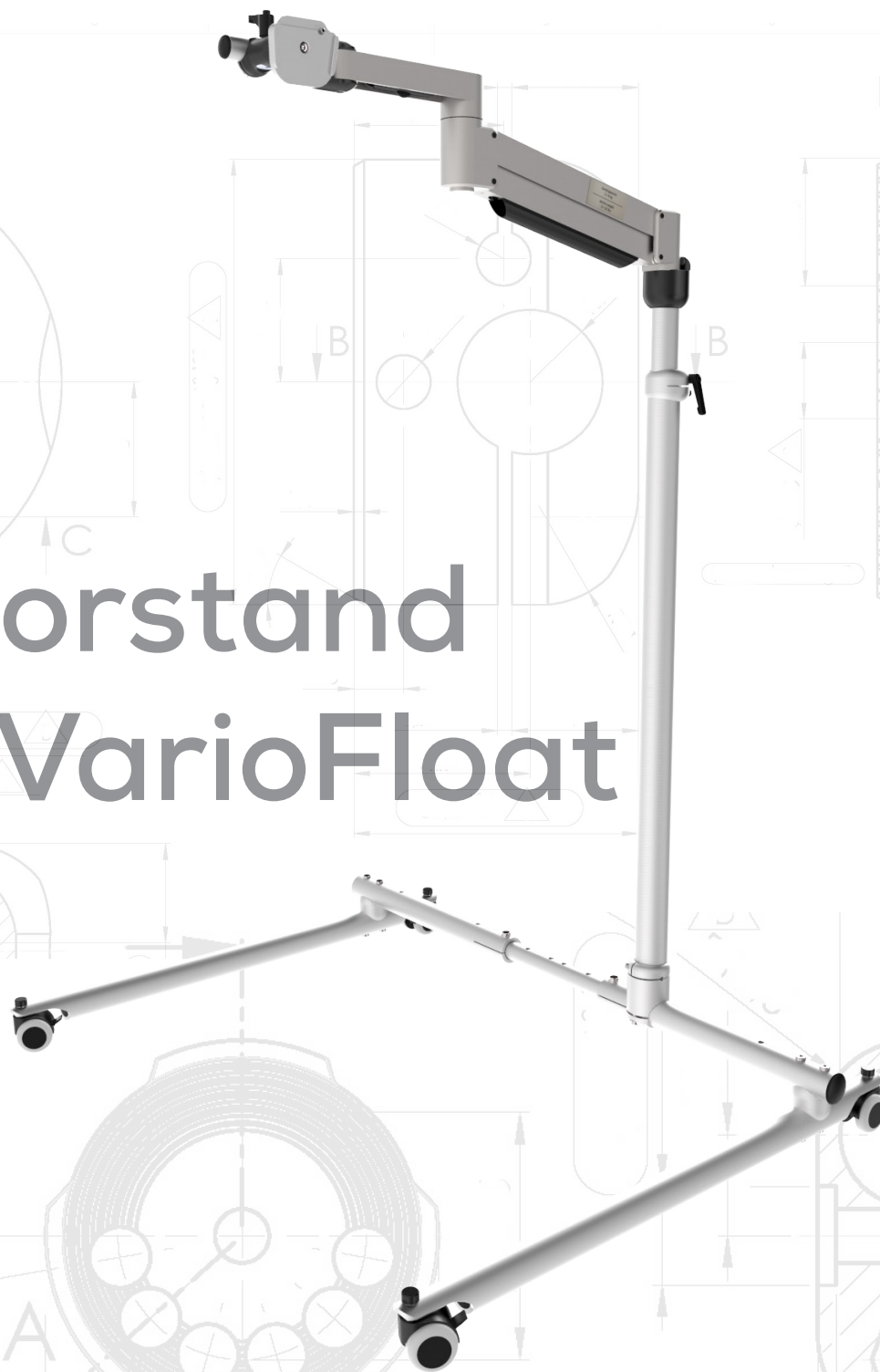
Name: Floorstand VarioFloat

Model Number: #17.1065

Instructions ID: #00.5061~F

## Floorstand VarioFloat

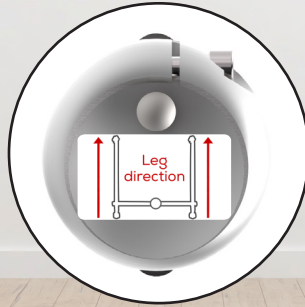
Rehadapt


## Attention / Achtung

When mounting the node, make sure the black arrow on the sticker points to the same direction as the legs.

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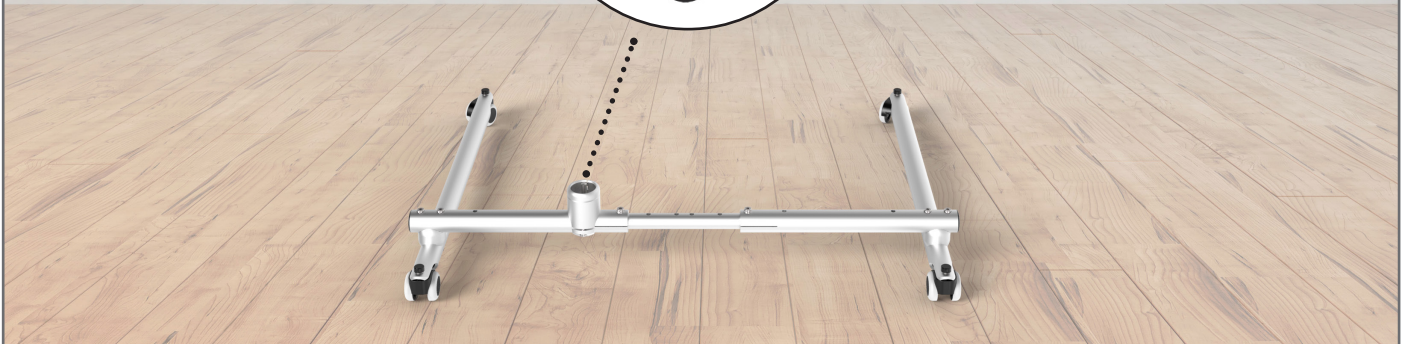
## Attention / Achtung


### Tilting hazard

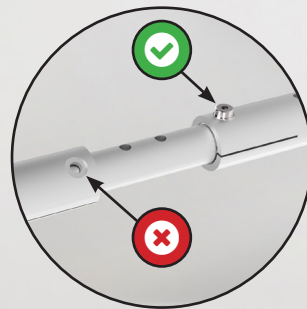Make sure that both screws for setting the width of the traverse are always present.

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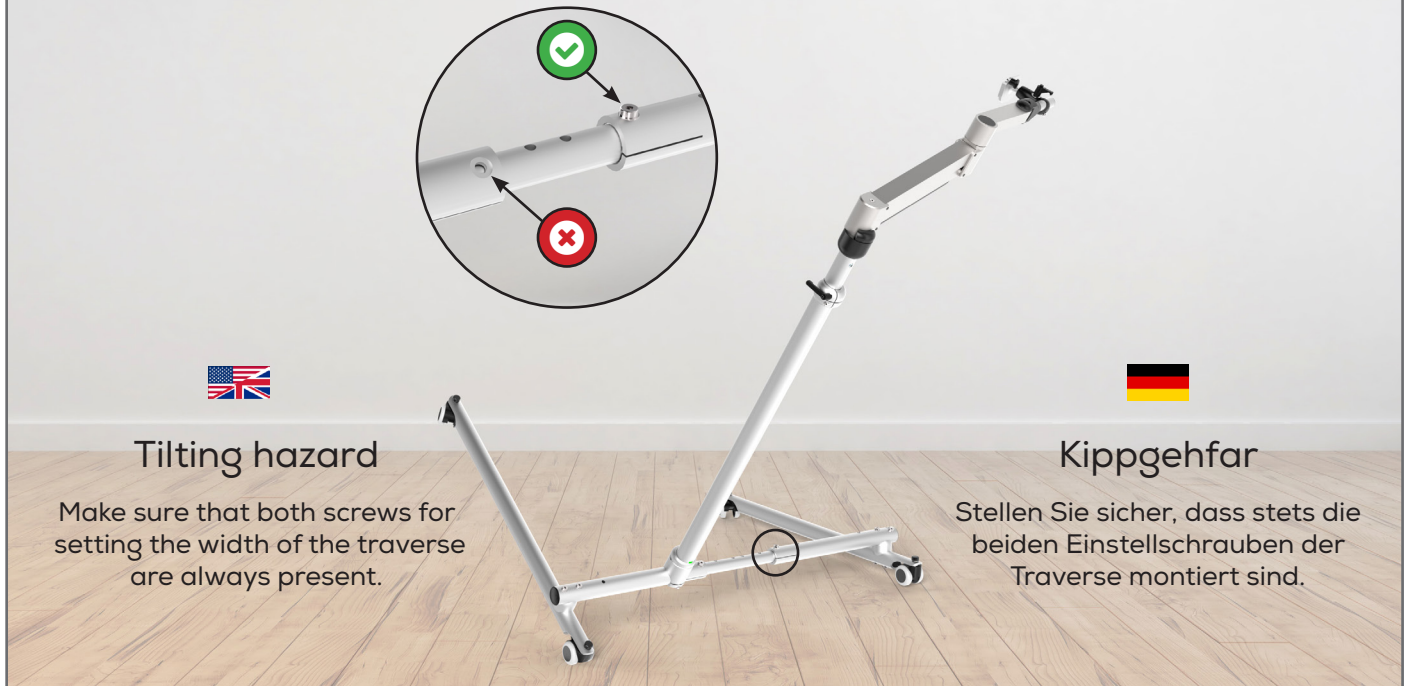
## Attention / Achtung

### Swinging Area

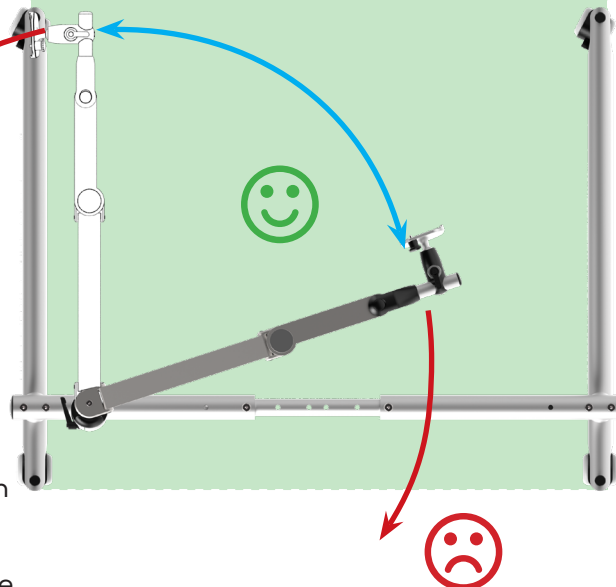Make sure that the column is oriented so the FS arm swings over the base of support illustrated with the blue arrow.

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## Attention / Achtung

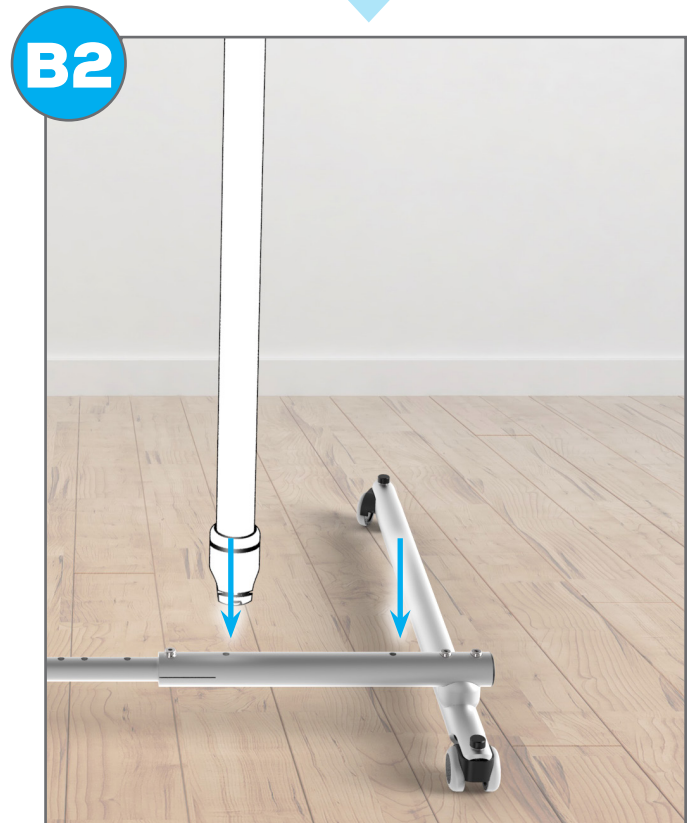
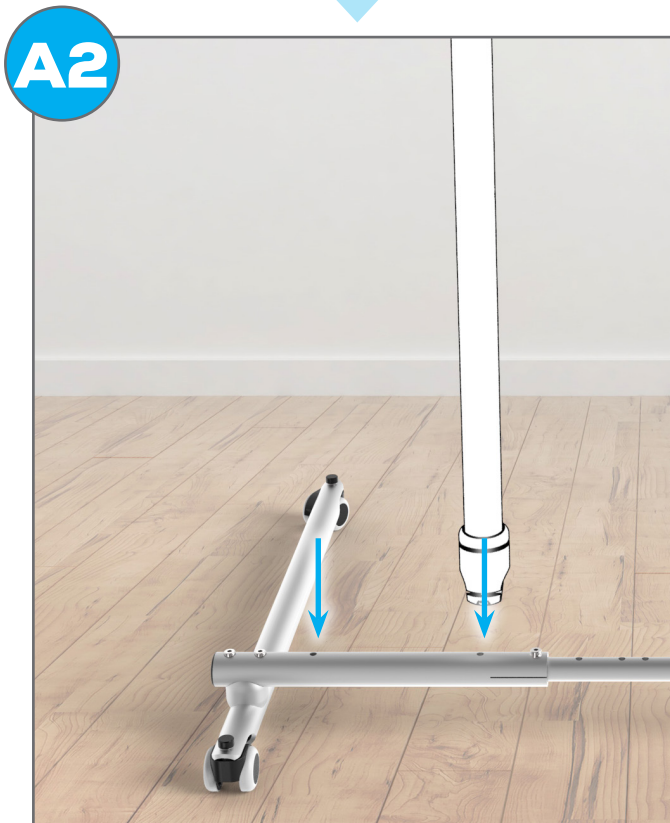
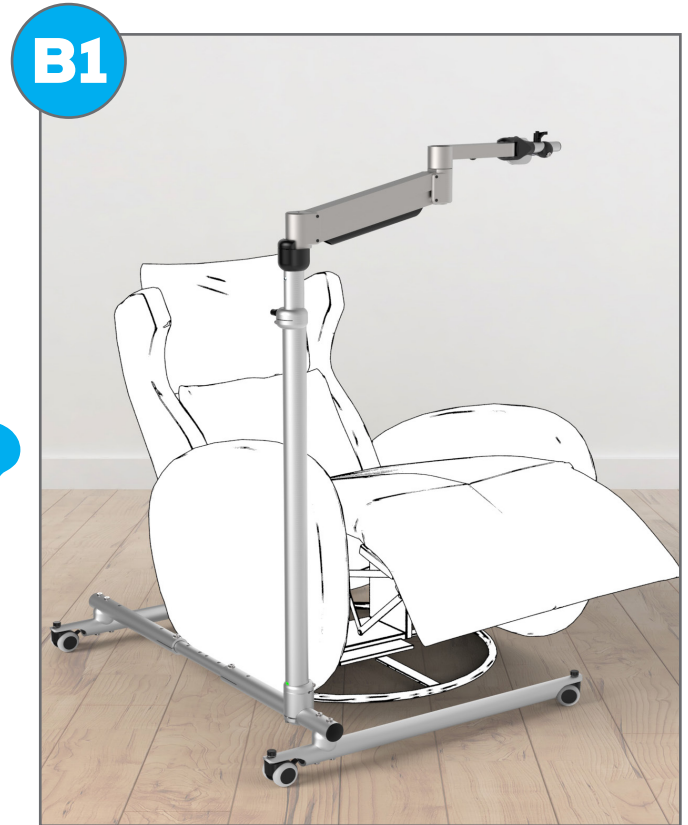
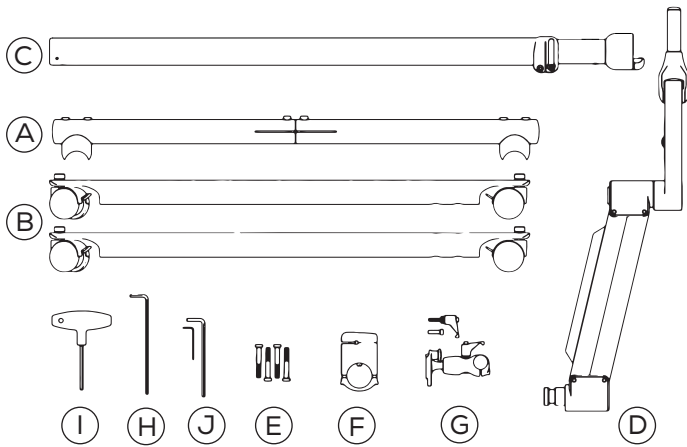
Before removing the device make sure the FloatArm is set on its highest position.

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
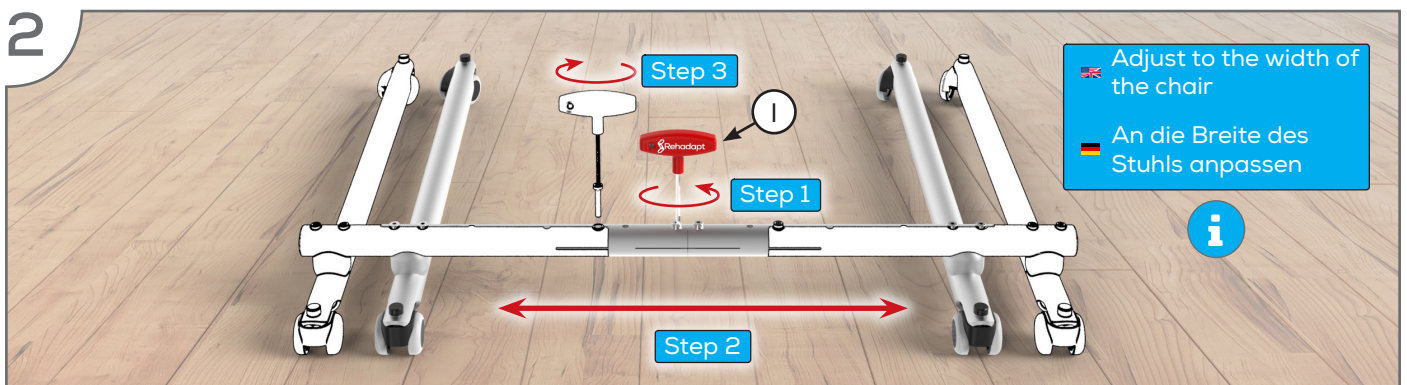
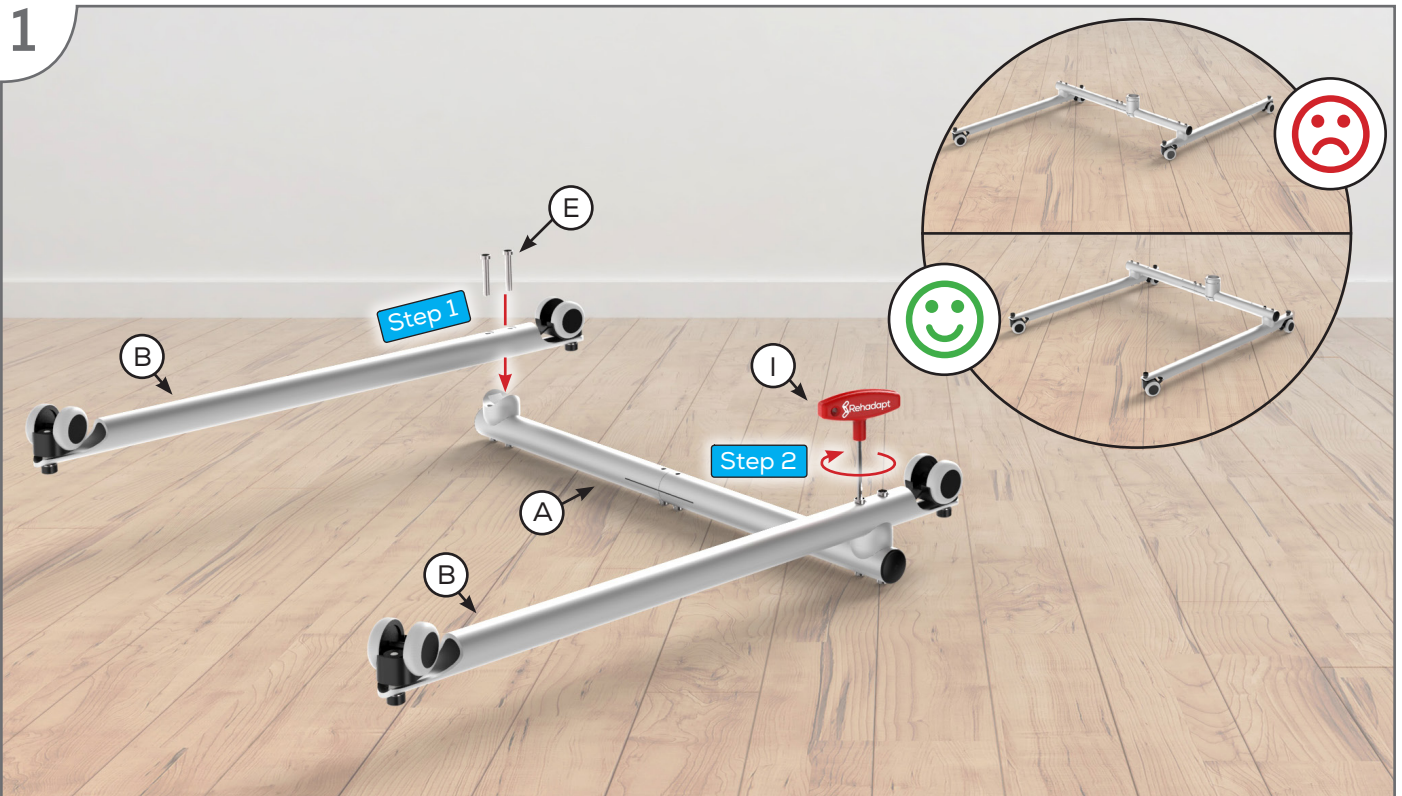
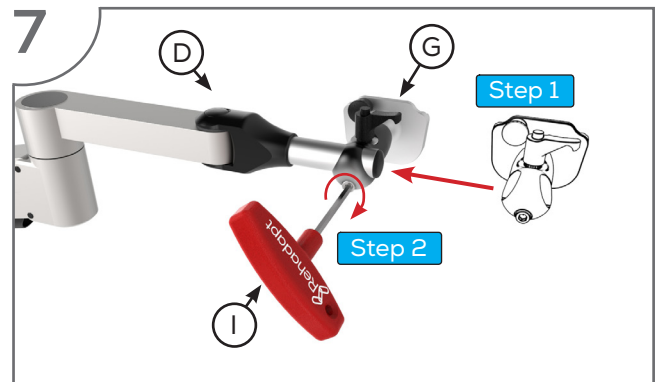
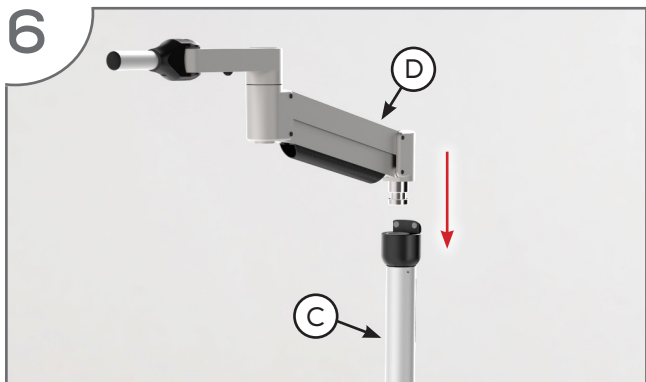
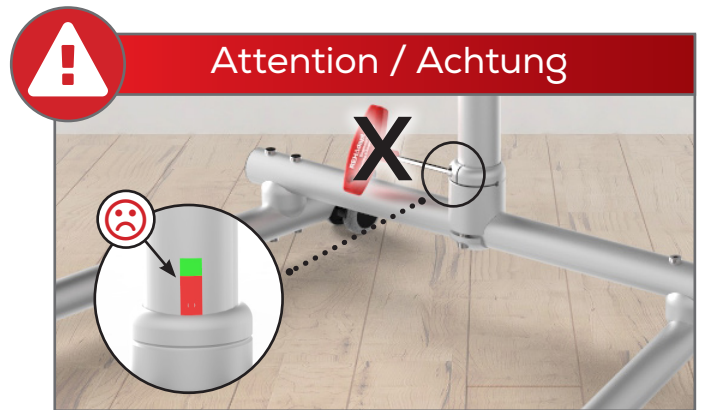
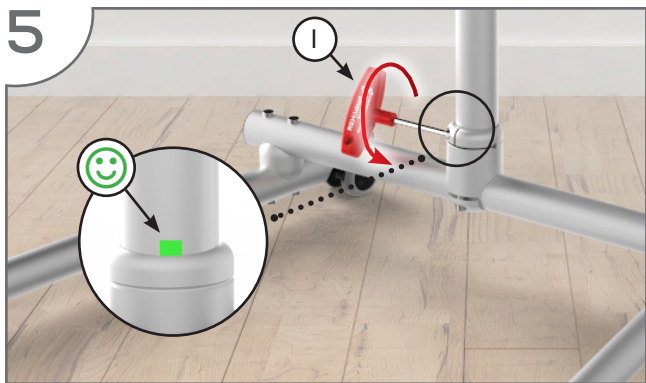
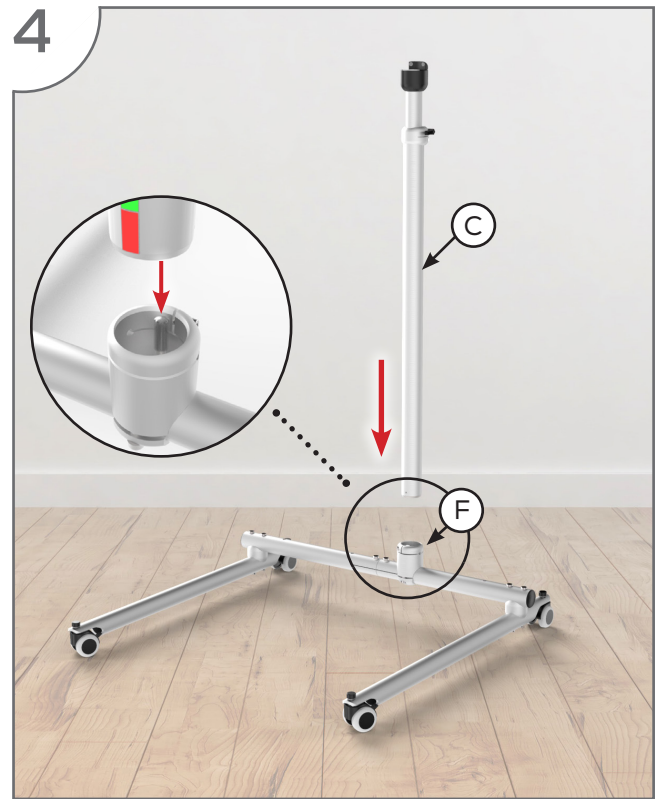
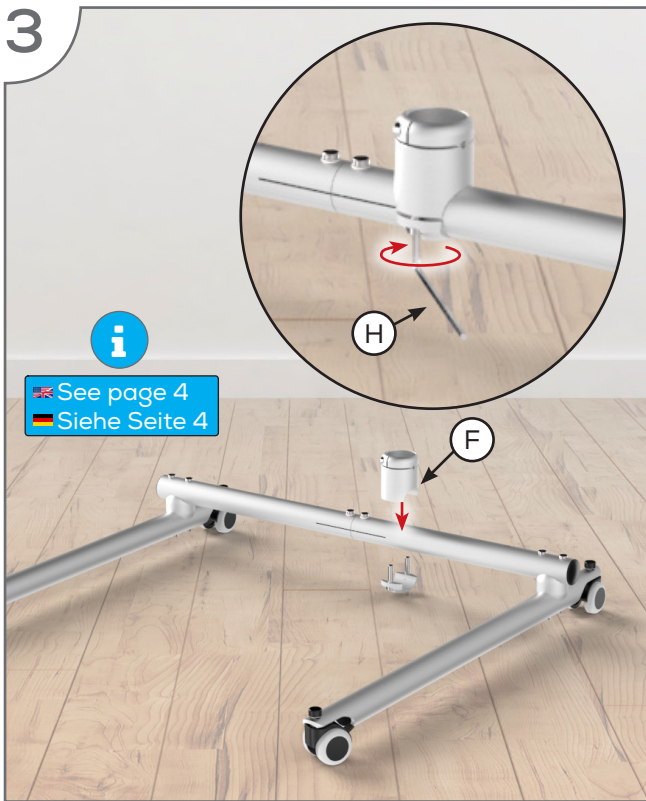
Instructions ID: #00.5063~C

## IMPORTANT!

- Monitor must be on arm
- Arm must be in mount

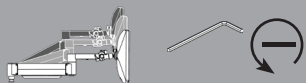**Press the arm down to approximately parallel** with the desk prior to turning the strength adjustment screw.


If the arm **drifts upward**, turn the adjustment screw clockwise.


If the arm **drifts downward**, turn the adjustment screw counter-clockwise.

Depending on the weight of your monitor, **you may have to make 15-20 turns**.

## Step 1

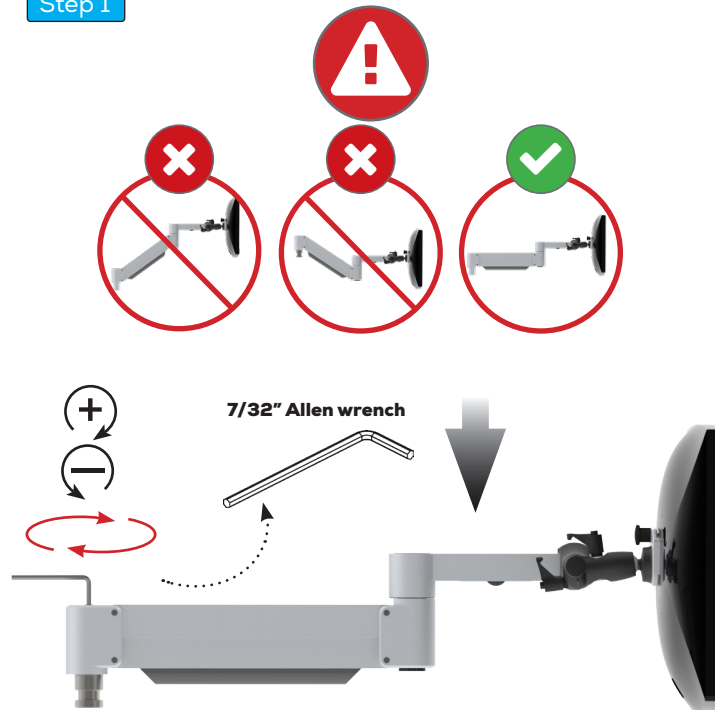
## Step 2

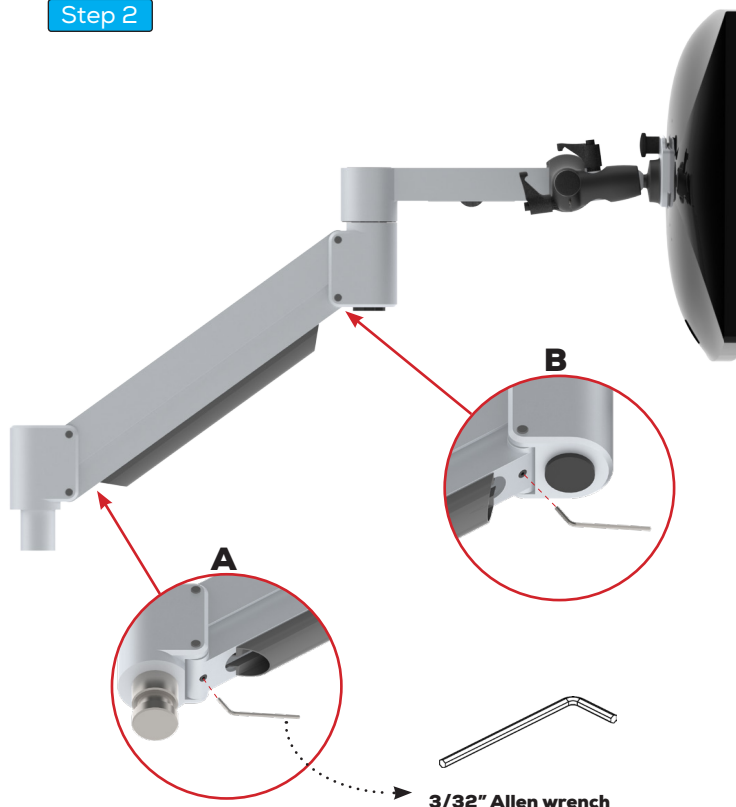**In some instances**, further counterbalance adjustments are needed to allow the arm's instant height adjustment function to work properly.

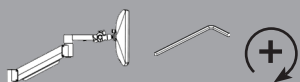If your arm is not staying in position after performing step 1, follow these steps:

With arm in raised position, tighten set screw **(A)**

With arm in horizontal position, tighten set screw **(B)**

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